

Thank you for purchasing  
this **REDGUM Brand** product.  
If you have any issues or questions about the  
Sock Aid please contact your supplier  
or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

### FEATURES

- Assist with putting socks on without having to bend
- Flexible sock holder contours to the foot
- Strong straps with loop handles
- Smooth inner lining / textured outer lining on sock holder

### SPECIFICATIONS

- Size: **875mm L x 180mm W** (when flat) x **3mm thick**
- Sock Holder: **225mm L**
- Strap: **650mm L x 25mm W**
- Loop Handle: **105mm L**
- Product Weight: **90g**

Contact the supplier or REDGUM Brand  
if more specifications are required.

### INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use the Sock Aid for its intended purpose which is being used as a device to assist with fitting a sock to a foot.


**INSPECT SUPPLIED PRODUCT** If you see any damage or suspect a fault, DO NOT USE THE PRODUCT and contact your supplier for support.

*Please turn over for more instructions*


**ASK FOR THE BRAND**

**redgumbrand.com.au**

**USAGE GUIDELINES**

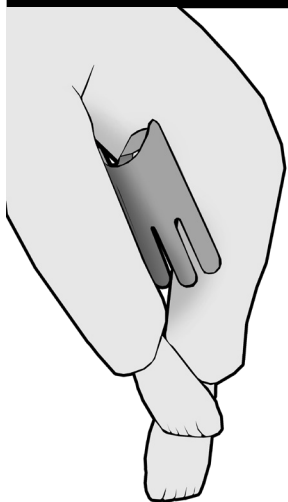

1. Place the sock holder in between your thighs and squeeze to hold sock aid in place.

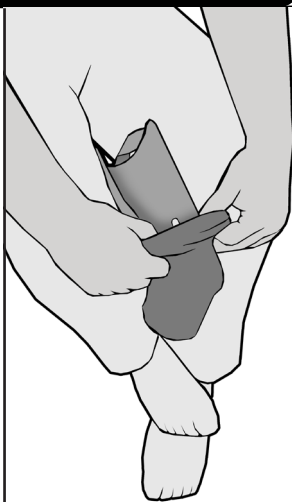


2. Bunch the sock up and slide onto the sock holder.




3. Pull the sock completely over the sock holder.



4. Holding the straps, place the sock holder on the ground in front of you and slide your foot in.

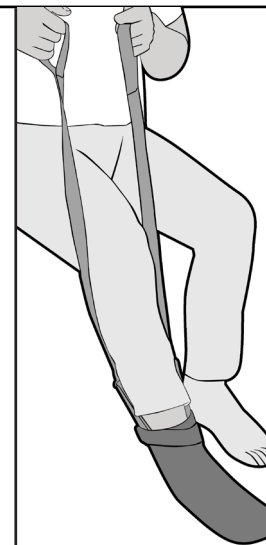

5. Using the straps, pull the sock holder onto your foot.

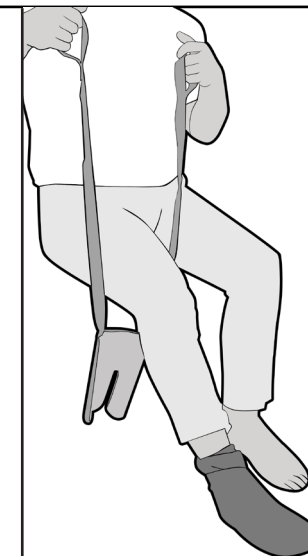

6. Once the sock is on foot, carefully slide the sock holder out behind your leg.

**INSPECTIONS**

We suggest that you check the Sock Aid visually periodically for signs of damage or wear prior to use. Withdraw the Sock Aid immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for support.

**CLEANING**

Clean the Sock Aid with warm soapy water if required and allow to dry prior to use. Abrasive cleaners or abrasive cleaning pads could damage the Sock Aid and should not be used.

**WARRANTY**

**Redgum Brand Sock Aid (RG71SA)** is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.