

Suction cups  
on underside  
of mat

### FEATURES

- Provides a secure non-slip surface in a wet area
- Textured surface for grip
- Drain holes allow water to escape
- Suction cups on underside

### SPECIFICATIONS

- Size: **550mm L x 550mm W x 5mm H** (approx.)
- Product Weight: **680g**

Contact the supplier or REDGUM Brand if more specifications are required.

Thank you for purchasing this **REDGUM Brand** product.

If you have any issues or questions about the Square Bath Mat please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180** E: **orders@for-de.com.au** A: **1 Business Way, Malaga, Western Australia 6090**

### INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances. Only use the Square Bath Mat for its intended purpose which is being used as a device to provide a non-slip surface in a wet area.

**INSPECT SUPPLIED PRODUCT** If you see any damage or suspect a fault, DO NOT USE THE PRODUCT and contact your supplier for support.

*Please turn over for more instructions*

**ASK FOR THE BRAND**


**redgumbrand.com.au**

## USAGE GUIDELINES

Ensure the surface with the suction cups is facing down to the floor.

Lay the mat out flat in the required location and press down to engage suction cups.

Test the mat to ensure it is securely held in place prior to use.

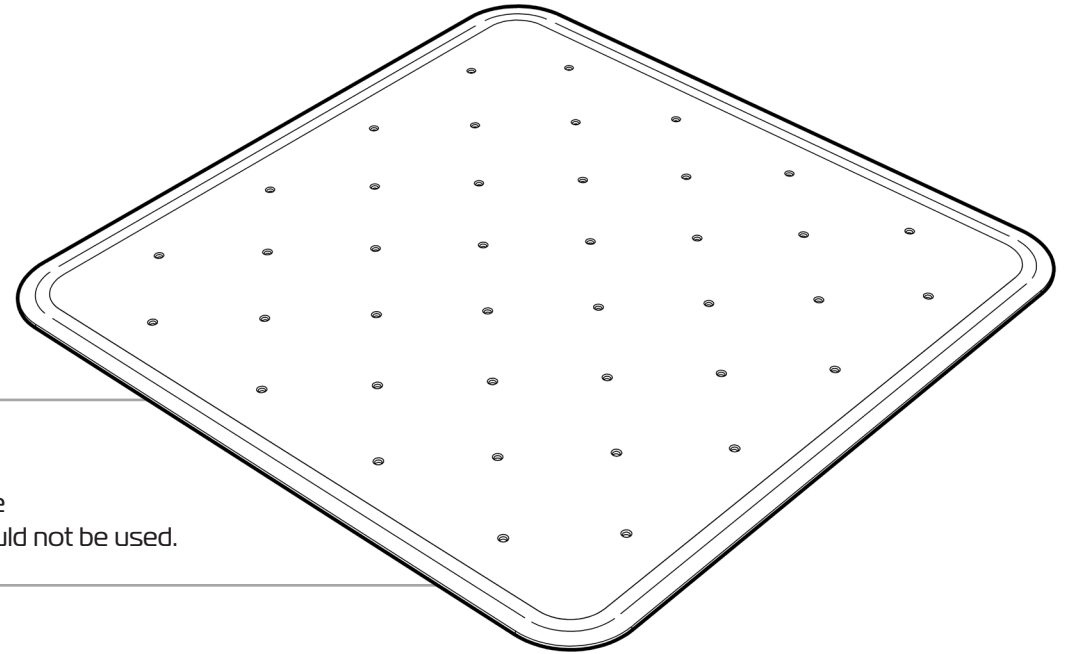

## CLEANING

Clean the Square Bath Mat with warm soapy water if required and allow to dry prior to use.  
Abrasive cleaners or abrasive cleaning pads could damage the Square Bath Mat and should not be used.

## INSPECTIONS

We suggest that you check the Square Bath Mat visually periodically for signs of damage or wear prior to use.  
Withdraw the Square Bath Mat immediately if you suspect damage or any other fault.  
If in any doubt, please do not issue or use, but immediately contact your supplier for support.

## WARRANTY

**Redgum Brand Square Bath Mat (RG71SBM)** is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment.  
Failures as a result of ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.  
Our goods come with guarantees that cannot be excluded under the Australian Consumer Law.  
You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage.  
You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.